SnapIT

Multi-Use | Ampoule Opener

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LITE Edition

SnapIT Lite comes in regular and large. The lightweight plastic components means you can add this to your lanyard with other items and not be weighed down.

- made from sturdy, low cost plastic.
- hexagonal anti-roll collar.
- great promotional item or gift.
- regular 1ml to 15ml ampoules, that's 70% of all ampoules used.
- large opens 5ml to 25ml ampoules.

HEXAGONAL COLLAR HELPS STOP ROLLING.

DON'T LET THIS HAPPEN TO YOU

THERE IS A SAFER WAY

Instructions for Use

INSERT

ALIGN DOTS:


DOTS

SNAP IT

AWAY FROM DOT:

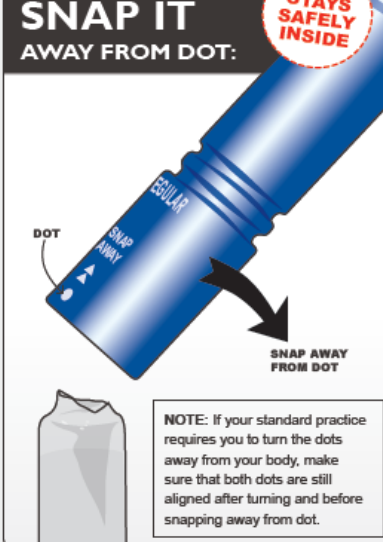LID STAYS SAFELY INSIDE


DOT

SNAP AWAY FROM DOT

NOTE: If your standard practice requires you to turn the dots away from your body, make sure that both dots are still aligned after turning and before snapping away from dot.

EJECT

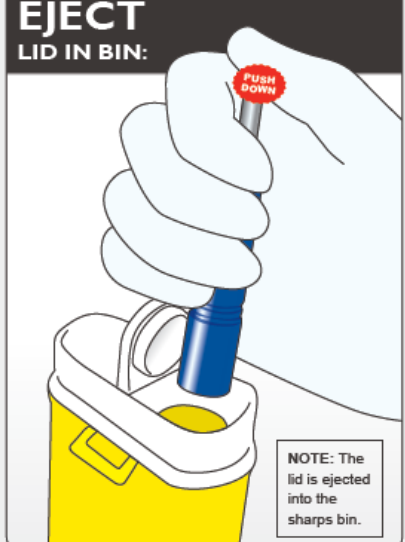LID IN BIN:

PUSH DOWN

NOTE: The lid is ejected into the sharps bin.

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Sizes

- If the ampoule lid is too big, it will not fit through the entrance. Use a larger Snapit.
- If the ampoule is too small, the Snapit will not hold the top securely. Use a smaller Snapit.

Ampoule Size	Snapit Size	Ampoule Size	Snapit Size
1 - 2 ml	REGULAR Most Common Sizes	Large 5 - 10 ml	LARGE
5 - 10 ml			
10 - 15 ml			
		10 - 15 ml	
		20 - 25 ml	

Snapit size is determined by Ampoule lid. Content in mls can be used as a guide.

Due to the inconsistent variation of tops, this is only a guideline.

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